



Stool Culture - New Specimen Container

Lawson #68519

Effective immediately, submit specimens for stool culture in Cary Blair media kits. Cary Blair media improves recovery of stool pathogens, such as *Campylobacter*, *Salmonella*, and *Shigella*.



Fill to line. Do not overfill!
Fill using a tongue depressor or collection spoon attached to lid.
Vigorously shake fluid vial.
Pink indicator fluid. Do not use if color fades or turns yellow.
Each kit contains: <ul style="list-style-type: none">• 1 vial with Cary Blair (green lid)• 1 empty vial (clear lid)• 1 set of collection instructions• 1 ziplock bag
Stability: 72 hours ambient or refrigerated

Please contact Nancy Madinger, MD at 720-848-7047 if you have any questions or visit our website at <http://www.uch.edu/for-healthcare-professional/Clinical-Laboratory/index.aspx> for additional information.

Issued July 25, 2011

Clinical Laboratory
Medical Director Ronald B. Lepoff, MD